# Pinnacle™

PEAK FLOW METER  
ISO 23747

## INSTRUCTIONS FOR USE

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**Fyne Dynamics Ltd**  
1 Horsecroft Place, Harlow,  
Essex CM19 5BT, UK  
+44 (0)1279 423423

info@fyne-dynamics.com

### About Your Peak Flow Meter

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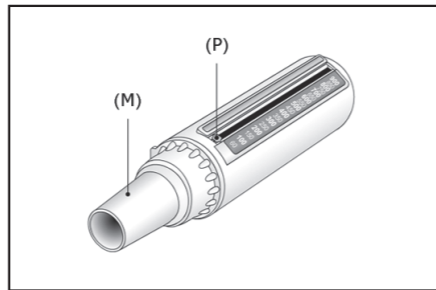
1

Ensure the white mouthpiece is clean. Insert the mouthpiece (M) into the meter and move the pointer (P) to the zero end of the slot. Lightly hold the meter so that you do not obstruct the slot or the holes at the end.

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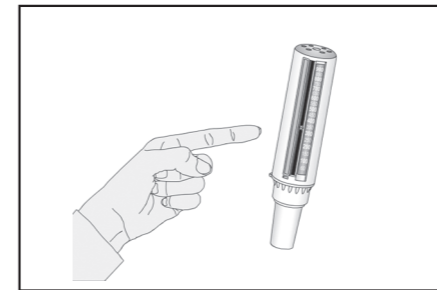
2

Standing up, or sitting up straight, take as deep a breath as possible and place the mouthpiece into your mouth, making a seal with your lips. Blow into the meter as fast and as hard as you can. Do not put your tongue into the mouthpiece. Do not use a spitting or coughing action, as this can cause false readings.

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
3

Take the meter from your mouth and note the number opposite the pointer. Your PEF is the highest of three attempts. Repeat the procedure twice more and note the number each time. Enter the highest number of the three on the chart overleaf.

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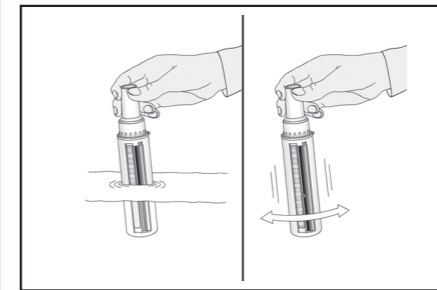**CLEANING** Your meter should be cleaned from time to time. Immerse the meter in warm mild detergent solution for 2-3 minutes and agitate. Rinse in clean warm water, shake off excess water and allow to air dry. DO NOT SUBJECT THE METER TO BOILING WATER.

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### Checking Your Peak Flow Meter / Troubleshooting

You should check your meter periodically to see if it is functioning correctly. Check for poor fit of the mouthpiece, cracking or discolouration. If unusual readings are obtained, this could be due to malfunction of the meter. Check that your technique is correct and try again. If the pointer does not move, it is broken, and the meter should be replaced. A damaged meter should not be used.

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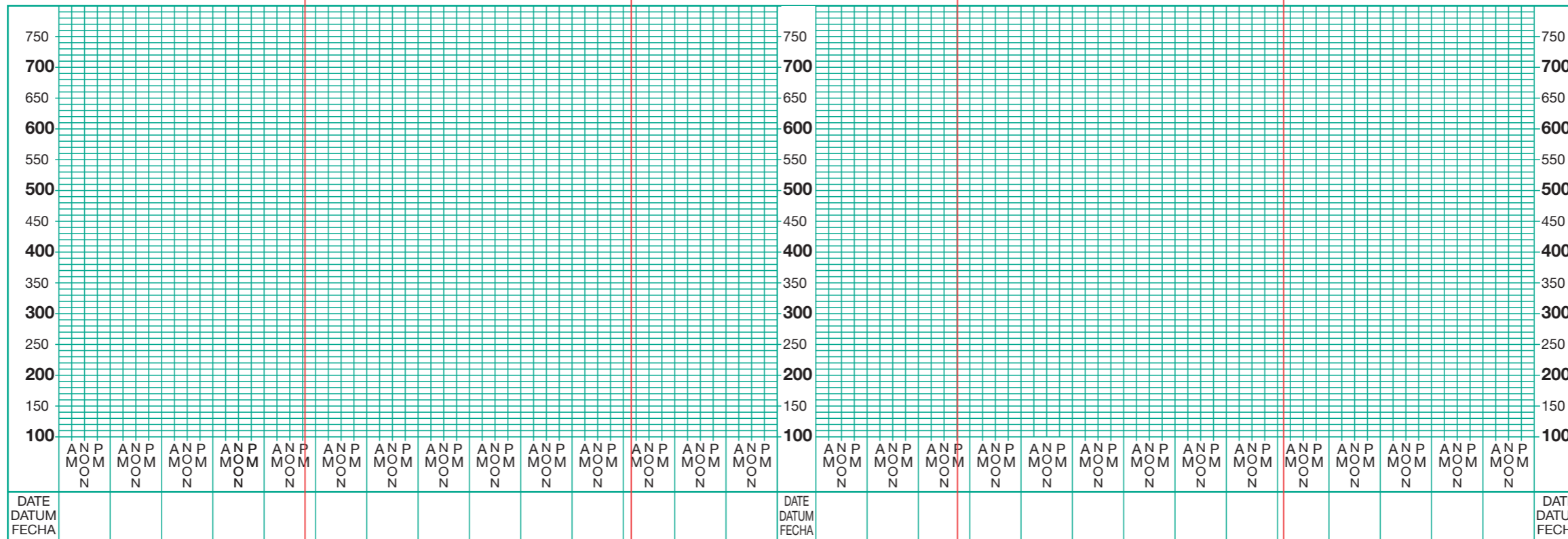
			<b>Date:</b> 25/03/20120
BLACK	PANTONE GREEN	CUTTER GUIDE (non printing)	<b>Flat Size:</b> 150 x 445mm
			<b>Folded Size:</b> 150 x 65mm

72041 PinnacleIFU Iss 4: 03/2020


You should record your PEF two or three times a day as directed by your doctor or practice nurse. Plot your PEF on the chart (above) every day and write in the date. Join up the points as shown. Remember that only the highest value of your three attempts should be plotted. Your doctor will need to examine your PEF chart periodically.

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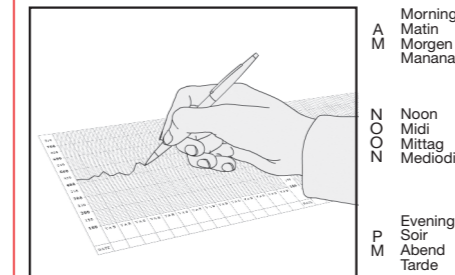**(GB) Taking Care Of Your Pef Meter**  
Keep the meter in a cool dry place, and do not expose it to excess heat. The detachable mouthpiece should be cleaned regularly in a mild detergent solution. Extremes of heat, humidity and altitude can affect the performance of the meter. The meter meets all International Standards and with careful use will give accurate performance for at least three years.

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